



STAYING ALIVE LLC

Your AED equipment needs monthly maintenance in order to be ready when you need it. Saving a life could depend on it!

Complete this Checklist for each AED each month.

- Dust AED carry case.
- Visually inspect case. Ensure there hasn't been tampering or damage.
- Unzip carry case to view AED. Ensure there hasn't been tampering or damage.
 - *** DO NOT PULL OPEN THE AED PADS ***
- Look for green light flashing in the display window. This is the battery indicator. Green means battery is strong and functioning.
- Verify the expiration date on the pads. Order replacements if expired.
- Zip carry case closed.
- Inspect attached prep kit (gloves, face shield, scissors, razor, drying pads). Ensure all items are clean and ready to use.
- Date and initial inspection card.

Should you see any damage or tampering with the case or unit or if the battery indicator is not flashing green, contact Staying Alive immediately for consultation. Call Staying Alive to order new pads.

Registration and maintenance of your AED is required by state law.